


DESIGNED FOR 1st PASS SUCCESS WITH ALL CLOT TYPES


POSITIONING

For multiple Drop Zones to interact with clot, deploy NeVa distally with the proximal or 1st Drop Zone markers at the leading edge of the occlusion.

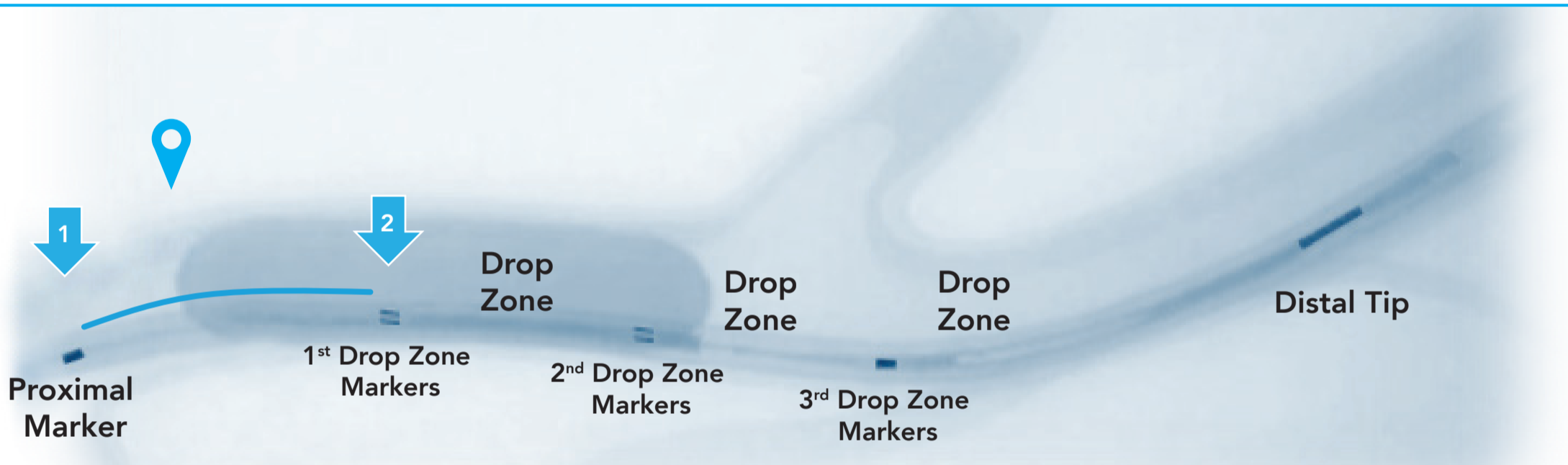

SIZE CHOICE

Solid lines = site of occlusion
Dotted lines = landing spot of distal portion of devices

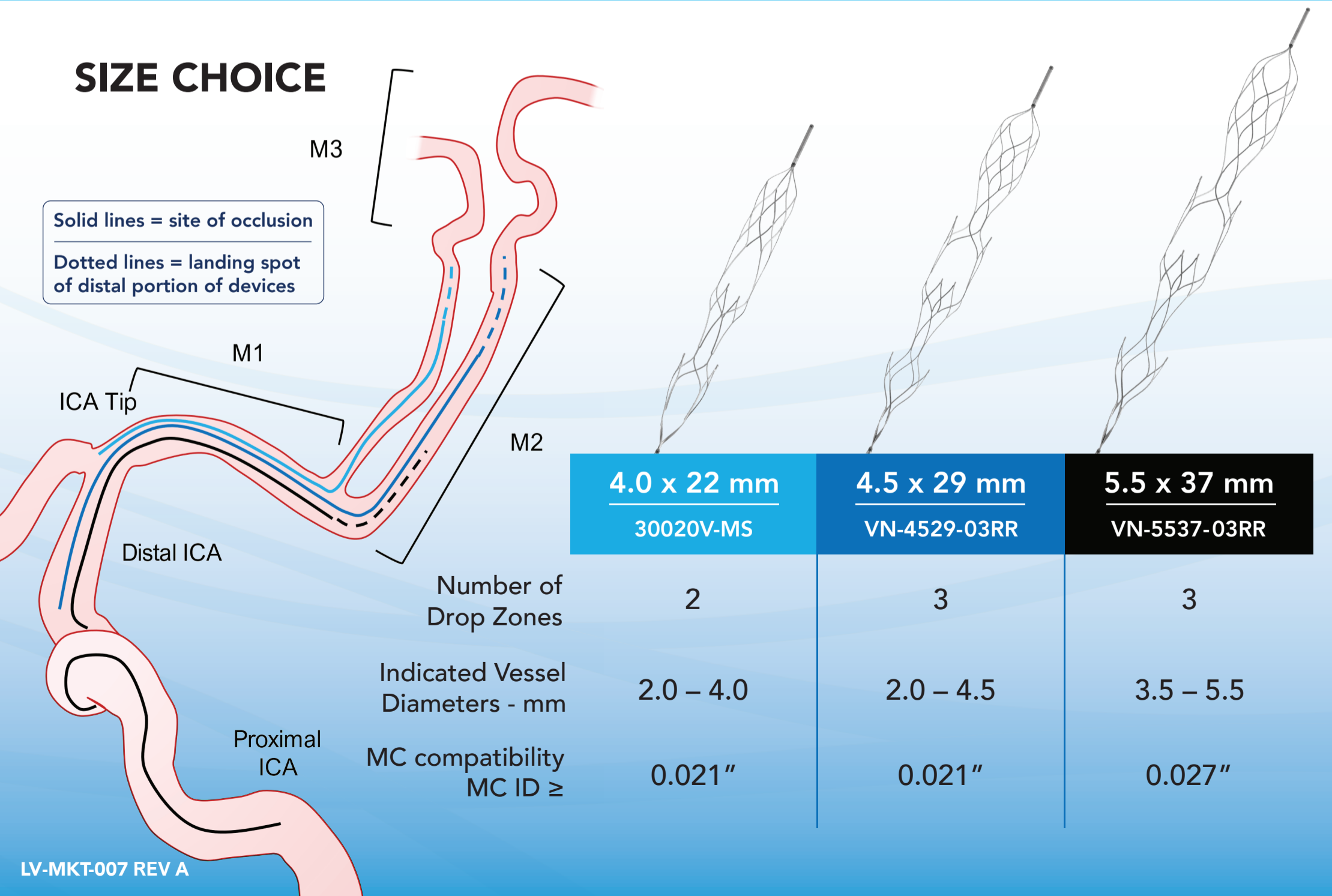

	4.0 x 22 mm 30020V-MS	4.5 x 29 mm VN-4529-03RR	5.5 x 37 mm VN-5537-03RR
Number of Drop Zones	2	3	3
Indicated Vessel Diameters - mm	2.0 – 4.0	2.0 – 4.5	3.5 – 5.5
MC compatibility MC ID ≥	0.021"	0.021"	0.027"