




WHEN COMPLEXITY REQUIRES SIMPLICITY

Designed to remove arterial
peripheral distal occlusions
with simplicity



pVasc[®] THROMBECTOMY SYSTEM PROCEDURE


SETUP

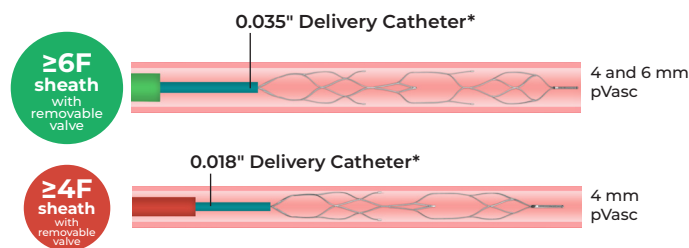

* Delivery Catheter is designated per wire compatibility.
Recommendation: Remove before retrieval.


OUT OF PACKAGE ASSEMBLY

pVasc is preloaded in an Introducer

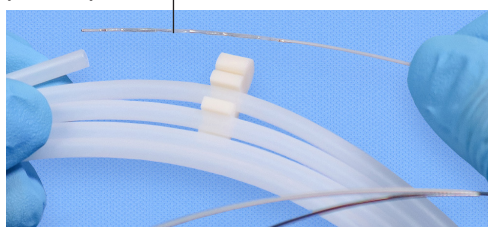

1 DELIVER

- Deliver wire distal to the clot (a)
- Ensure delivery catheter is positioned distal to clot (b)
- Remove wire
- Prep pVasc with heparinized saline and advance pVasc until Striped Markers on Pusher Wire are completely in Introducer. Remove Introducer. If using 0.035" delivery catheter, hold onto Introducer while advancing pVasc
- Under fluoro, continue advancing pVasc until Distal Tip aligns with delivery catheter exit marker

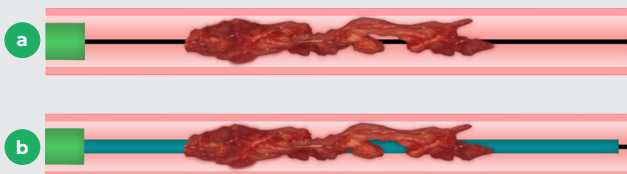

2 DEPLOY

- Deploy pVasc under fluoro by retracting delivery catheter while securing pVasc Pusher Wire (c)
- Check positioning
- Remove delivery catheter (d)



3 RETRIEVE

- Retrieve pVasc under fluoro
- Apply aspiration on sheath as pVasc enters and exits sheath
- Remove cross-cut valve from sheath once distal portion of Pusher Wire is visible

RADIOGRAPHIC MARKERS



PROCEDURE
VIDEO



POSITIONING

Proximal marker aligns with proximal edge of clot



PRODUCT SPECIFICATIONS

Product Name	Product Code	Diameter (mm)	Working Length (mm)	Full Length (mm)	Wire Length (cm)	Number of Drop Zones	Recommended Vessel Diameter (mm)	Delivery Catheter	
								ID	Guidewire Compatible
pVasc 4.0 x 22 mm	VP-4022-02RR	4.0	22	39	200	2	≥2.0 & ≤3.5	0.021"	0.018"
pVasc 4.0 x 30 mm	VP-4030-F2RR	4.0	30	48	200	2	≥2.0 & ≤3.5	0.027"	0.035"
pVasc 4.0 x 38 mm	VP-4038-F3RR	4.0	38	55	200	3	≥2.0 & ≤3.5		
pVasc 6.0 x 44 mm	VP-6044-F3RR	6.0	44	63	200	3	≥3.5 & ≤6.0	0.027"	0.035"

Indications: The pVasc® Thrombectomy Device is indicated for the non-surgical removal of emboli and thrombi from peripheral blood vessels; temporary use in peripheral vessel occlusion; use with aspiration and with the injection or infusion of contrast media and other fluids.

Prescription use only. Please see www.vesalio.com for indications for use, contraindications, warnings, precautions, and potential adverse events. For complete information, including detailed instructions for use, please refer to the Instructions for use.

© Vesalio, Inc. 2026. All rights reserved. The name Vesalio and the product names and associated logos are registered and unregistered trademarks of Vesalio, Inc. The following trademarks are registered with the USPTO: Vesalio®, NeVa®, NeVa VS®, NeVa NET®, and pVasc®; and with the EUIPO and UKIPO: Vesalio™, Drop Zone™, NeVa™, NeVasc™, pVasc™, and enVast™.

VESALIO, INC.
5001 Spring Valley Rd.
Suite 120E
Dallas, TX 75244
Customer Service:
(800) 362-6985
LV-MKT-066 Rev C



www.vesalio.com